


# Needleless Transfer Device (NTD) Instructions for Use




Step 1

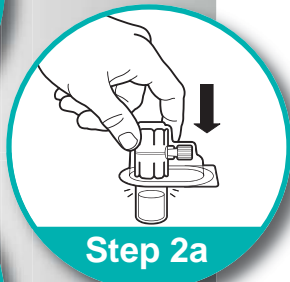

Step 2a



Step 2b

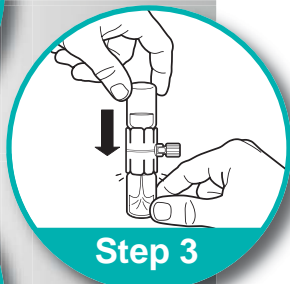

Step 3




Step 4




Step 5a




Step 5b

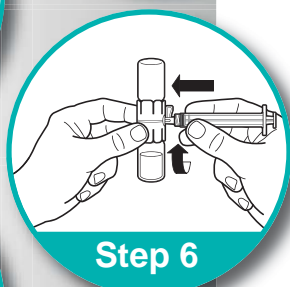


Step 6

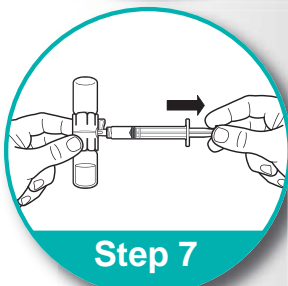

Step 7

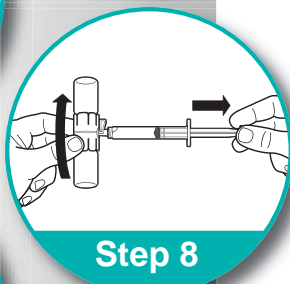

Step 8

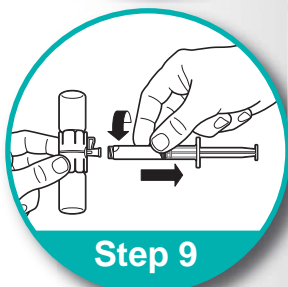

Step 9

Before handling the components, wash your hands with soap and hot water.

Place the package on a clean, flat surface.

Remove the Flip-Off® buttons from the sterile water and drug product vials.

Disinfect the injection sites with an alcohol pad.

## Step 1:

Peel the cover off the needleless transfer device pack.

Do not remove the device from the pack.

## Step 2:

Place the device on top of the diluent vial, using the blister pack as a holder. Seat the device on the diluent vial by pushing down until the spike penetrates the rubber stopper and the device snaps in place.

Remove the needleless transfer device from the blister pack and discard the package.

## Step 3:

Invert the diluent vial with the needleless transfer device attached; seat the open end of the device on the drug vial by pushing down until the spike penetrates the rubber stopper and the device snaps in place. The diluent will flow into the drug vial.

## Step 4:

Gently swirl the vial to make sure the product is thoroughly mixed.

## Step 5:

Turn the valve on the side of the device toward the drug vial and remove the cap.

## Step 6:

Remove the syringe's protective cap. Connect the syringe to the luer lock by turning it clockwise.

## Step 7:

Draw back on the plunger, and then inject air into the vial.

## Step 8:

Turn the system so the drug vial is on top.

Pull back on the plunger to withdraw the drug.

## Step 9:

Remove the syringe from the device.

The drug is now ready for administration.

Follow normal safety practices to administer the drug.